


## 3 in 1 commode

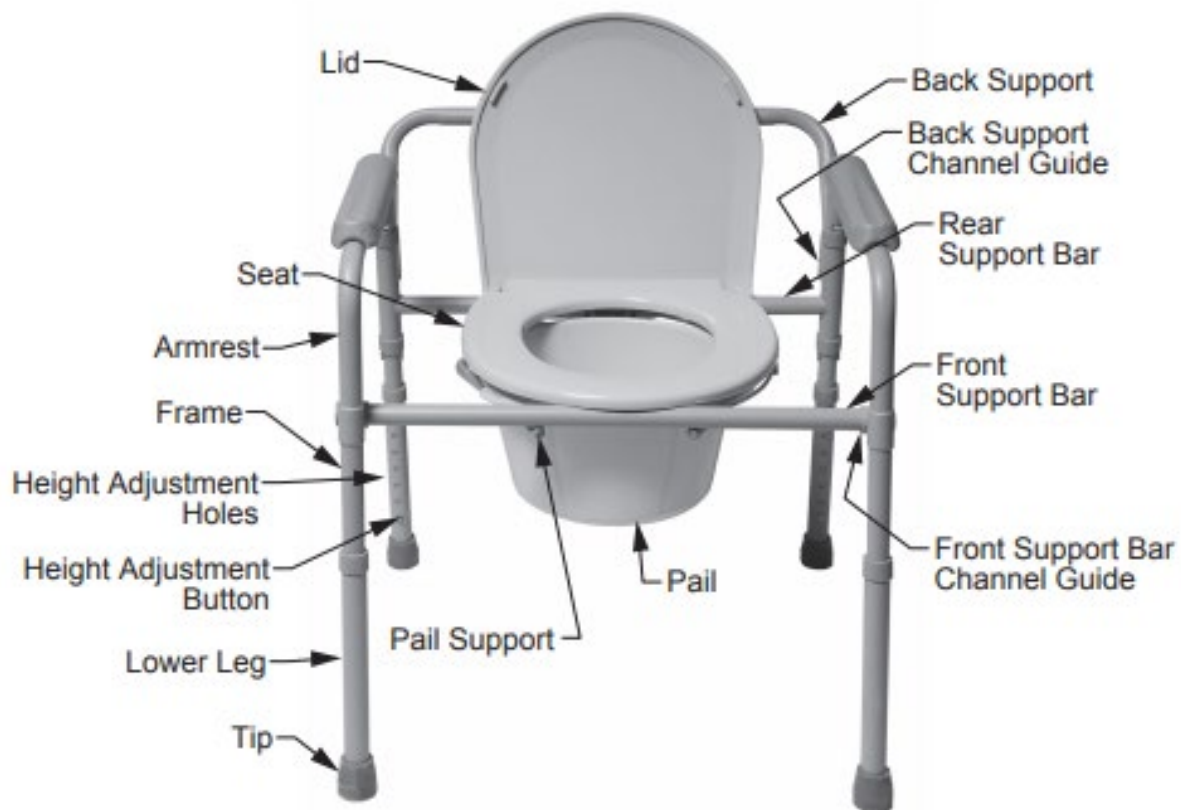

(example of a typical 3-in-1 commode)

### Adjusting Commode Height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button clicks into place in the desired hole.
2. Repeat for other 3 legs.


**WARNING:** Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

## How to use the commode over a toilet


1. Set commode to desired height.
2. Remove commode pail and lid.
3. Position splash guard in place on pail supports.
4. Remove back support (if necessary)
5. Lift toilet bowl seat.
6. Place commode over toilet with splash shield positioned over the center of the toilet bowl.

Commode is now ready for use. Clean the seat and splash guard with disinfectant after each use.

In use over  
the toilet for  
arm support

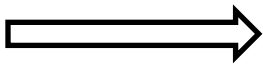

Frame of a  
typical  
commode

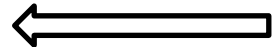

## How to use as a bedside commode

1. Set commode to desired height.
2. Lift commode seat and lid and place commode pail on pail supports. Ensure that pail is properly positioned below commode opening.
3. Place commode seat down. Commode is now ready for use.
4. When user is finished using commode, place commode pail lid over commode pail to contain odors.
5. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant